
medoVital

Natur-Moor | natural moor

● **Wärmekissen**
heat cushion

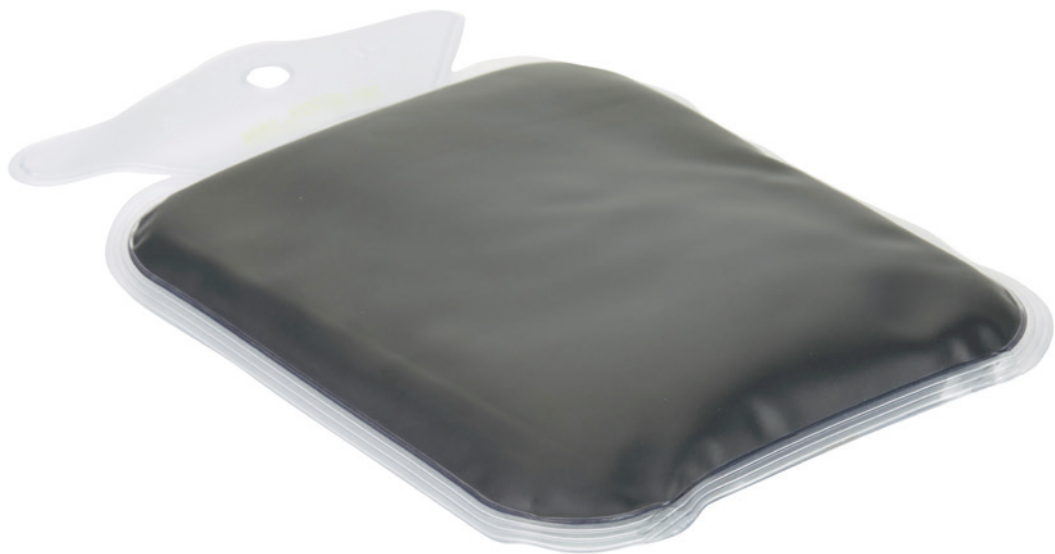● **Nackenkissen**
neck cushion

● **Rückenkissen**
back cushion

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GB INSTRUCTIONS FOR USE

Thank you for purchasing a natural moor cushion from MedoVital. This product is a medical product. It provides gentle healing and relaxation with natural ingredients. The natural moor cushion releases therapeutic warmth over a period of several hours.

The moor filling is a modified natural product. Colour variations and discolorations occur naturally. The moor is obtained from regenerative extraction in Germany. The film used is free of critical softeners. The natural moor cushion is produced in the Lower Rhine region of Germany.

Please note the following information. This will ensure long-term use of your high-quality natural moor cushion. Please contact your doctor or pharmacist if you have any questions.

INSTRUCTIONS FOR USE

Carefully read the instructions in full, because they contain important information. This medical device is available without prescription. However, the natural moor cushion must be used as prescribed to ensure the best possible treatment results.

Keep the instructions in a safe place. You may wish to refer to them later.

COMPOSITION

Moor cushion: Cover = PVC, contents = modified natural

FORM OF ADMINISTRATION AND CONTENTS

1x natural moor cushion, 1x instructions for use

AREAS OF APPLICATION

The natural moor cushion is designed for home use and is ideal for heat and cold therapy, as it can be reused many times. It is recommended for the following complaints:

Indications for heat therapy	Local heat therapy for pain relief, reducing tension and for improved circulation. Proven to be effective as a supporting treatment for: Lumbago, sore muscles, back pain, abdominal/stomach pain, menstrual discomfort, neck pain, tenseness, stiff neck and joints, night-time calf cramps, rheumatic complaints (Please ask your doctor or pharmacist about use; do not use for rheumatic inflammation!).
------------------------------	---

The muscles relax and the regenerative capacity of the body is increased at a temperature of 40 - 43° C. The increased blood flow stimulates the metabolism, promotes the natural healing process and enhances the general sense of well-being. Prolonged warmth relieves pain!

Indications for cold therapy	Local cold therapy for the relief of pain and swelling. Proven to be effective as a supporting treatment for: Bruises, sprains, strains, headache, migraine, fever, toothache, insect bites, nosebleeds, blood effusions, oedemas, tennis elbow, inflammation of muscles and joints, refreshment of tired and heavy legs, rheumatic complaints (Please ask your doctor or pharmacist about use!).
------------------------------	--

Coldness alleviates inflammation, since the reduced body temperature inhibits the activity of the inflammatory mediators. To begin with, vascular reactions are triggered and the muscles tense during cold therapy. The sensation of pain is reduced. If used for a longer period, muscular activity is decreased, muscular tension is reduced and cramps are relieved.

Note: The symptoms may have various causes. You should consult your doctor if there is no improvement.

CONTRAINDICATIONS

Local thermal therapy should only be used on uninjured skin.

The following contraindications are stated for local thermal therapy:

Do not use in the following cases:

- People who cannot feel heat or cold properly
- People requiring care and children under 3 years
- **Simultaneous ingestion of painkillers**
- Temperature sensitivity disorders
- Circulatory complaints
- Injured skin

Do not use cold treatment in the following cases:

- Arterial circulatory disorders and high blood pressure
- High sensitivity to cold
- Serious systemic diseases
- Rare diseases, in which cold treatment can cause harm, such as cold agglutinin disease, Raynaud syndrome, scleroderma, vasculitis or cryoglobulinemia
- Acute bladder and kidney symptoms
- Angina pectoris (relative)
- High blood pressure (relative)

Do not use heat treatment in the following cases:

- High temperature
- Acute arthritis
- Gout
- Peripheral arterial occlusive disease from Stage 3
- Increased tendency to bleed
- Tumour diseases
- Decompensated cardiac insufficiency
- Feverish infections
- Sensitivity disorders

No other restrictions known if used as prescribed. If in doubt, ask your doctor or pharmacist.

INTERACTIONS WITH OTHER MEDICINES

The skin must be free of gels, grease, ointments and essential oils to prevent allergic skin reactions. **Painkillers and other pharmaceuticals may impair temperature perception. The simultaneous ingestion of painkillers and other pharmaceuticals before or during heat or cold treatment could cause burns.**

SIDE EFFECTS

None known so far. Inform your doctor or pharmacist if you experience side effects Always seek medical advice if your symptoms worsen or there is no improvement.

Note: Please inform the manufacturer or the relevant authorities in the case of serious side effects.

Precautionary measures and information of special note:

No special precautionary measures are required if used as prescribed.

What do you have to consider during pregnancy and breastfeeding?

There are no concerns associated with use during pregnancy and breastfeeding.

PREPARATION FOR USE**Use in heat therapy:**

	Moor heat cushion	Moor neck cushion	Moor back cushion
Microwave (at max. 600 watts) Stop every two minutes and knead the cushion	Heat for approx. 2-3 minutes, continue in 10 second intervals if required	Heat for approx. 2-3 minutes, continue in 10 second intervals if required	Heat for approx. 3-4 minutes, continue in 10 second intervals if required
Oven (Set to a max. 120° C) Do not preheat, place on a non-metallic surface	Heat for approx. 10 minutes, continue in 2 minute intervals if required	Heat for approx. 10 minutes, continue in 2 minute intervals if required	Heat for approx. 15-20 minutes, continue in 2 minute inter- vals if required
Water bath Place in a sufficiently large vessel with hot, not boiling water	Heat for approx. 12 minu- tes, continue in 2 minute intervals if required	Heat for approx. 12 minu- tes, continue in 2 minute intervals if required	Heat for approx. 15 minu- tes, continue in 2 minute intervals if required

Carefully check with the back of your hand that the natural moor cushion has reached a comfortable temperature all over (40-43°C). Wrap the natural moor cushion in a towel before use.

Use in cold therapy:

Precooling in the refrigerator:

Place the natural moor cushion in the refrigerator for an unlimited period (approx. 5°C) for cold treatment. Permanent storage in the refrigerator possible. Wrap the natural moor cushion in a towel before use.

Note: The application time of 10 minutes should not be exceeded for cold therapy.

SAFETY INFORMATION:

Risk of injury!

- Handle with care after heating. The specified heating and precooling times must be strictly complied with.
- Overheating the natural moor cushion can cause burns. Bear in mind that temperatures as low as 48°C can cause burns if used for a longer period of time. Always wrap the natural moor cushion in a towel.
- Temperature fluctuations may occur at certain points during heating, so knead the natural moor cushion before use and distribute the heat evenly.
- The product is not a toy. It may only be handled by adults.
- Do not leave the natural moor cushion unattended during heating. If the natural moor cushion inflates, the heating temperature has been exceeded. In this case, do not touch the natural moor cushion with your bare hands! Switch off the microwave or oven immediately and check the product for damage after it has cooled down. The cushion may not be used in case of visible damage, which can be caused by overheating, pointed and sharp objects or by impermissible impacts (e.g. falling on the floor)! No liability is accepted for any consequential damage.
- The natural moor cushion may not come into contact with hot metal while being heated in the oven.
- Do not cut open the natural moor cushion. Do not swallow the contents.
- Never place the heated or cooled natural moor cushion directly on the skin. Wrap the natural moor cushion in a towel.
- Do not use during sleep.
- **Store in a dry place and out of reach of children.**

STORAGE AND CARE

The natural moor cushion can be reused many times. Its service life depends on the frequency of use, care and storage. Clean the natural moor cushion with a damp cloth after use and allow to dry. Do not use any aggressive cleaning agents or solvents.

We recommend that the natural moor cushion is stored in the refrigerator or in a cool and dry place which is not exposed to sunlight together with the instructions for use.

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bb med. product GmbH
MedoVital Handels GmbH